

Cath Strap Attachment Instructions


Step 1 – Attach Base Unit directly to the Table

- A. Remove Table Mat.
- B. Snug white Base Unit to table. Place solid strap under table (Velcro ends on table top).
- C. Velcro ends cross to meet exactly.



Step 2 – Strap Velcro should meet exactly

- A. Straps positioned with male Velcro strap first.
- B. Female Velcro strap adheres to male the full length & width of straps.
- C. Press down firmly along strap overlap to assure good adherence. Blue Label will be on left side of table.
- D. Replace Table Mat

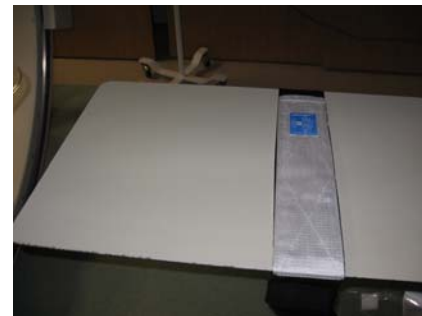

Step 3 – Proper positioning on patient

- A. Locate strap at approximate mid-chest level of patient by sliding along the long axis of the table.
- B. Wrap patient snugly keeping IV site accessible and removal tab free.
- C. **Note: Position should always permit free table movement (panning).**



Tip – Between cases

- Leave system on table. Simply slide toward head or foot of table.
- Take care straps do not touch floor – attach straps together and hang loosely under table.

