



F&H Innovative Products, LLC
PO Box 280
New Albany, Ohio 43054

Ph: 800.591.6002
Fax: 800.591.6002


Cath Strap - Tips

Proper attachment to of Base Unit to table permits free table panning

Keep IV accessible at all times

Place a sheet or towel between strap & patient to increase patient comfort

Wrap patient & overlap Velcro closure a minimum of 10"

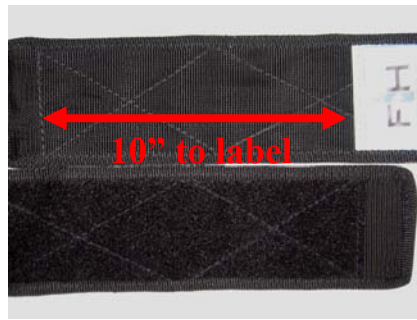

Leave system on table between cases

Take care straps do not touch floor – attach straps together and hang loosely under table

Simply slide toward head or foot of table for Table Cleaning



Strap System slides easily along length of table to adjust for patient height or table cleaning

Clean straps with ordinary soap & water or standard disinfectants used to clean table

Inspect regularly for integrity of seams & closures; replace System when visibly worn