

Open Rail Leg Strap Attachment Instructions

Step 1 – Attach Blue label strap to the table rail

- A. With the Blue label facing you, slip strap through the table rail on working side of table
- B. Pass 4" strap through the attached loop on the opposite strap end



Step 2 – Pull the 4" strap away from the table to snug a cinch knot at the rail



Step 3 – Attach White label strap to the table rail

- A. With The White label facing you, slip strap through the table rail on nursing side of table
- B. Pass 4" strap through the attached loop on the opposite strap end
- C. Pull the 4" strap away from the table to snug a cinch knot at the rail



Position strap across desired area of patient

- To close press Velcro firmly; to release pull tab

